

## Instructions for Creating a LogTag Profile


### *Or to Program Back-up LogTag*

\*Log Tags that are labeled '**Refrigerator**' or '**Freezer**' are programmed and ready to use.


\*You only need to add a profile to a '**Back up**' Log Tag **WHEN** or **IF** you need to use it.

**Example:** You would add a refrigerator profile only if you need an additional Log Tag to transport vaccine **OR** you need to replace an old nonfunctioning '**Refrigerator**' log tag

1. Open Log Tag software
2. Select **'LogTag'** and then **'Profiles'**

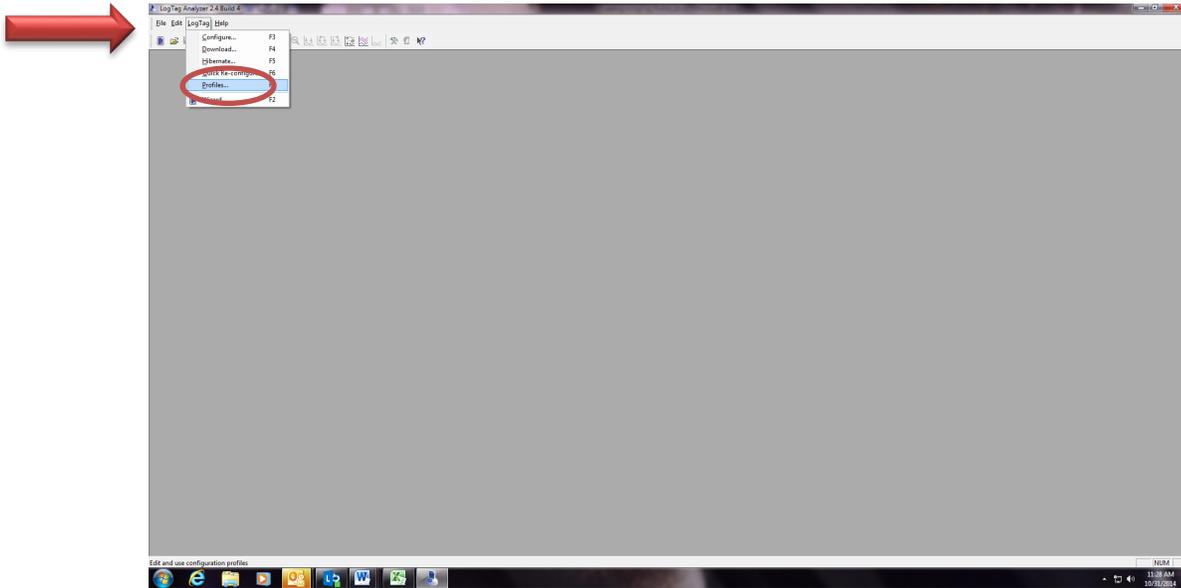


3. If box is empty, select **'New'** (if profiles already loaded start at 13 below)

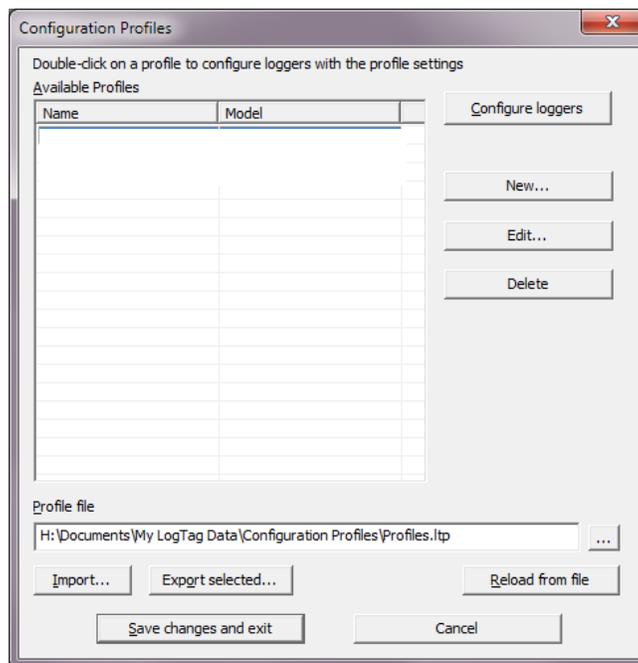


To add a Refrigerator Profile:


4. Add 'Refrigerator' to Profile Name and choose and set selections below

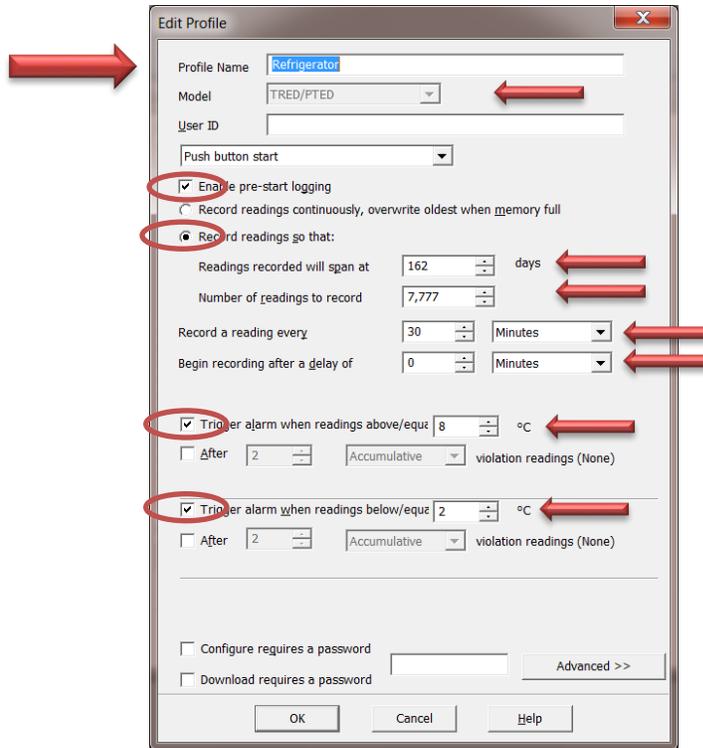


5. Select the 'Advanced' button

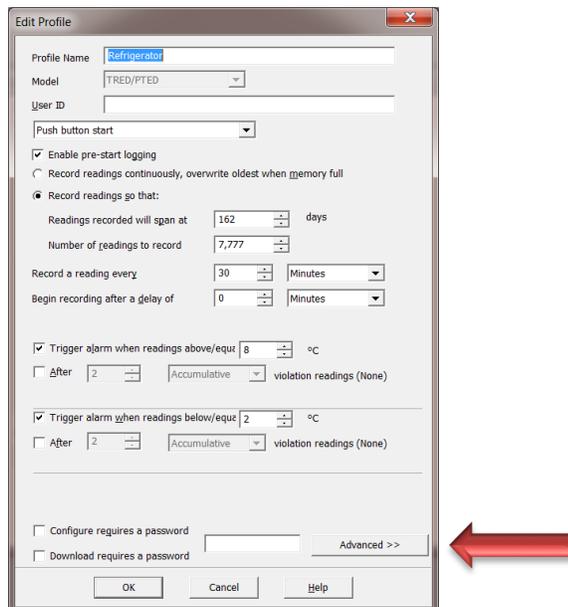


6. Next select the following settings

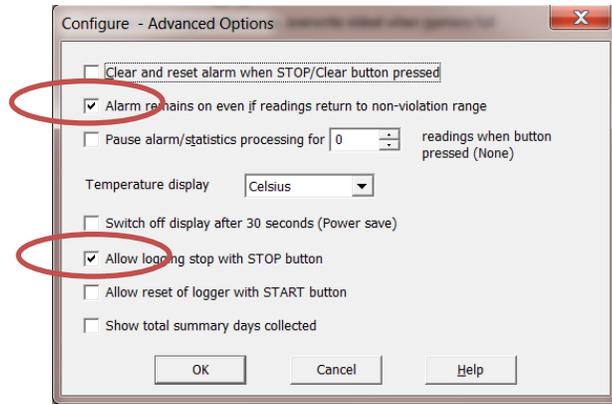

7. Click 'OK' to get back to the previous screen

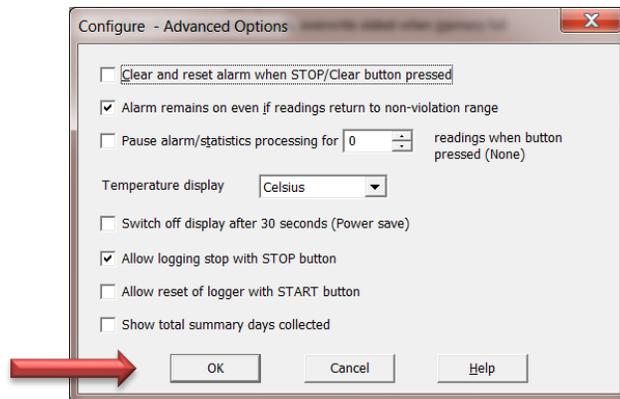

8. Click 'OK' again

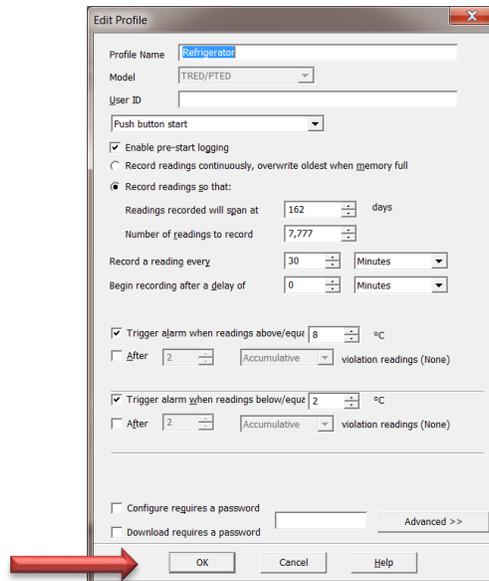

10. Next to add a **Freezer** profile follow the above instructions starting at 4 and ensure that the profile fields are the same as below

Profile Name: Freezer  
Model: TRED/PTED  
User ID:   
Push button start:   
 Enable pre-start logging  
 Record readings continuously, overwrite oldest when memory full  
 Record readings go that:  
Readings recorded will span at: 162 days  
Number of readings to record: 7,777  
Record a reading every: 30 Minutes  
Begin recording after a delay of: 0 Minutes  
 Trigger alarm when readings above/equal -15 °C  
 After 2 Accumulative violation readings (None)  
 Trigger alarm when readings below/equal -40 °C  
 After 2 Accumulative violation readings (None)  
 Configure requires a password  
 Download requires a password  
OK Cancel Help


Clear and reset alarm when STOP/Clear button pressed  
 Alarm remains on even if readings return to non-violation range  
 Pause alarm/statistics processing for 0 readings when button pressed (None)  
Temperature display: Celsius  
 Switch off display after 30 seconds (Power save)  
 Allow logging stop with STOP button  
 Allow reset of logger with START button  
 Show total summary days collected  
OK Cancel Help

Configuration Profiles  
Double-click on a profile to configure loggers with the profile settings  
Available Profiles  


Name	Model
Freezer	TRED/PTED
Refrigerator	TRED/PTED

  
Profile file: H:\Documents\My LogTag Data\Configuration Profiles\Profiles.ltp  
Save changes and exit Cancel


11. Next download and reconfigure your LogTag unit as normal and leave it in the docking station
12. When the download and reconfiguration is complete, close the data window (the window that opens with the downloaded data) and press the F7 key.
13. Highlight the configuration profile appropriate for the LogTag unit you are working with (see arrow below)

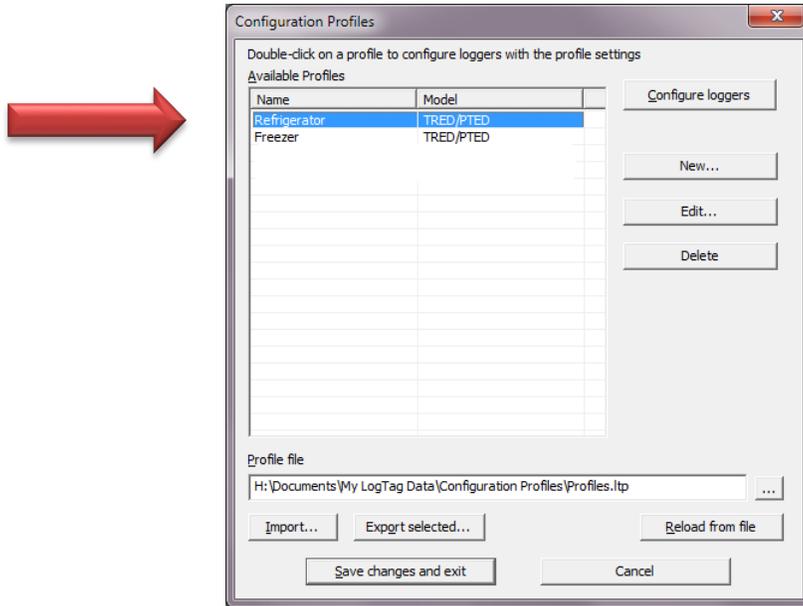

14. Next press the configure loggers button then save and exit

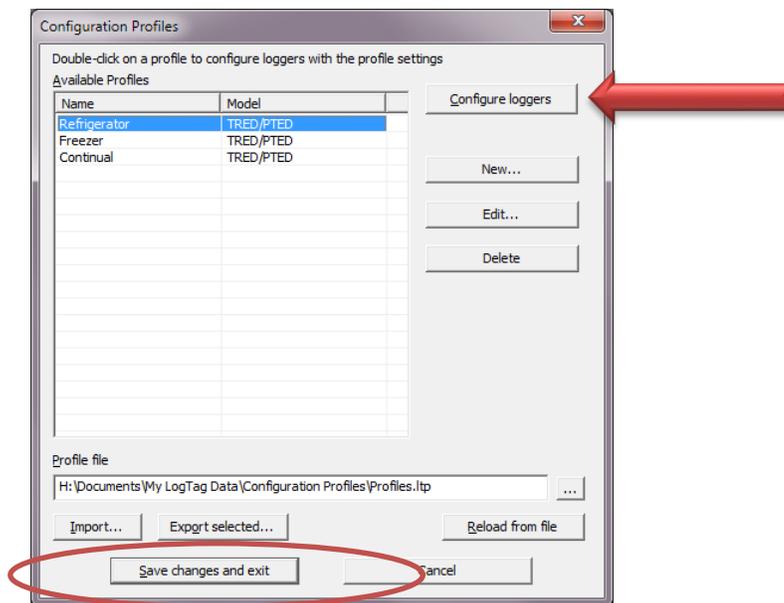

\*This will program your LogTag with the updated profile. Once it is indicated that the LogTag is ready to start/be started you can press the close button (see arrows below)

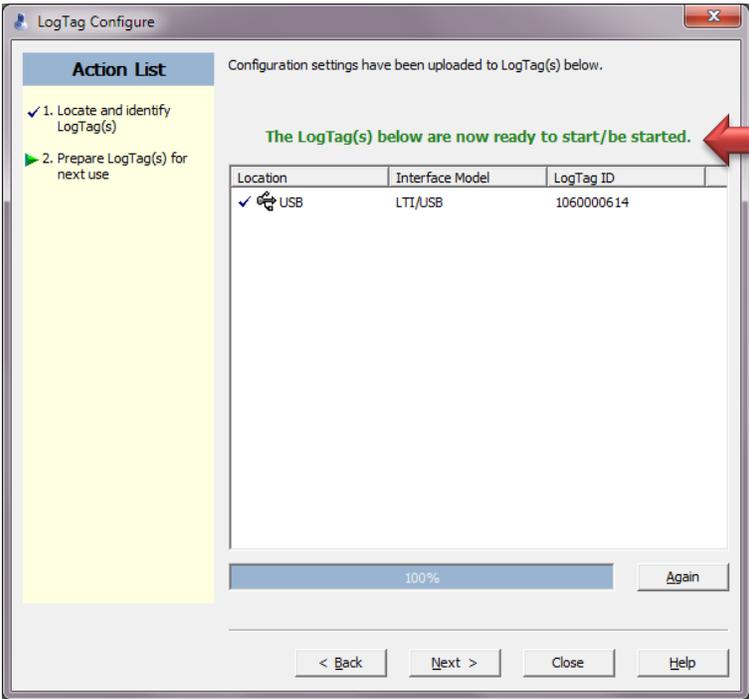

The LogTag is now ready to be started. Repeat the process with your remaining LogTag units starting with Step 13.